

DELUXE CONNECTED BLOOD PRESSURE MONITOR UA-651BLE

- 1) Install the included batteries into your blood pressure monitor. (BPM)

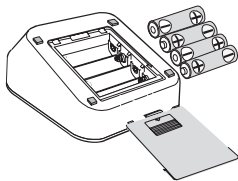- 2) Insert the air connector plug into the air socket.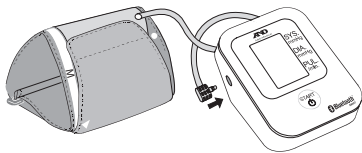

- 3) Download the free "A&D Connect" app from the App Store or Google play.


- 4) Turn on *Bluetooth*® settings on your mobile device (smartphone or tablet).

5) Install and launch the “A&D Connect” app. The app will guide you through the account set up and connection process.


The app will show you the following:

6) Press and hold the START button until “P_r” is displayed, then release the button.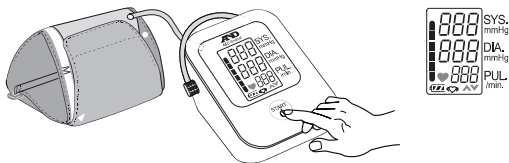


7) While “P_r” is shown the BPM is discoverable.

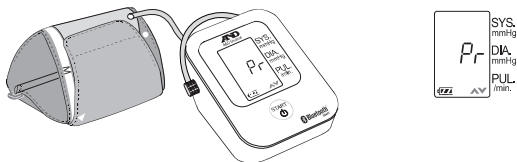8) While “SEt” is shown the BPM is pairing with your mobile device.


9) The BPM displays "End" to indicate the pairing process is complete.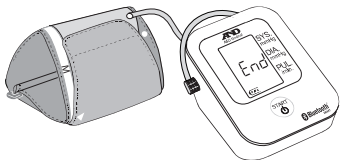


10) The blood pressure monitor is now ready to be used.

11) Lay your left arm flat on the table with palm up. Place the cuff on your upper arm.

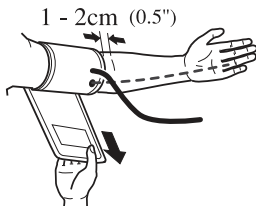
12) Fasten the cuff tight enough so you can slide two fingers underneath it.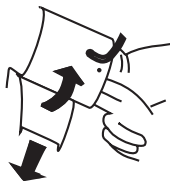


13) Sit comfortably with your arm resting on a flat surface, elevated to heart level, palm up.

14) Press the START button.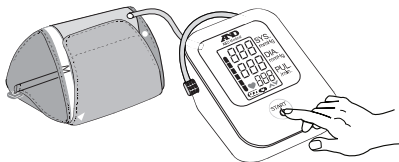

- 15) Sit quietly as cuff inflates and deflates.
- 16) Wait for the measurement to complete.
- 17) When the measurement is complete, the cuff deflates automatically.
- 18) Your systolic, diastolic pressure and pulse rate appear on the screen and measurements are sent to your mobile device when the "A&D Connect" app is open and running. Results are stored in the BPM when your mobile device is not available.

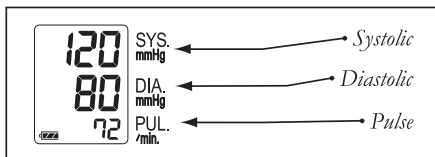BLOOD PRESSURE TAKING TIPS

- Compare blood pressure taken at the same time each day.
- Sit in the same chair/position.
- Do not cross legs and keep your feet flat on the floor.
- Relax for several minutes before measurement.
- Use the correct cuff size to get an accurate reading.
- Sit still during measurement-no talking, eating or sudden movements.

AND
A&D Medical

1-800-726-9966
www.andmedical.com

©A&D Medical. All rights reserved. iPad, iPhone and iPod are trademarks of Apple, Inc. Android and Google play are trademarks of Google, Inc. Specifications subject to change without notice.

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